RADIFOCUS *Torque Device*

Multi-Torque Vise

POIGNEE DIRECTIONELLE

Usable Guidewire diameter / Diametre du guide : 0.010"-0.038"

ENGLISH

INDICATIONS FOR USE

RADIFOCUS Torque Device is intended to facilitate guidewire manipulation during interventional procedures.

INSTRUCTIONS FOR USE

Slip the RADIFOCUS Torque Device over the exposed end of the guide wire.
When the RADIFOCUS Torque Device is in the desired position on the guide wire, tighten the nut to clamp the handle in place.
The RADIFOCUS Torque Device may be repositioned or removed by loosening the nut and retightening it.

PRECAUTIONS

- Read instructions for use before use.
- Single use - Do not resterilize or reuse.
- Sterile in unopened, undamaged package.
- Do not use the RADIFOCUS Torque Device if its package is broken or soiled. The RADIFOCUS Torque Device should be used immediately after opening the package and be disposed of properly after use.
- AVOID EXPOSURE TO WATER, DIRECT SUNLIGHT, HIGH TEMPERATURES, OR HIGH HUMIDITY DURING STORAGE.

Rx ONLY CAUTION : Federal law (U.S.A.) restricts this device to sale by or on the order of a physician.

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Do not resterilize
Ne pas restériliser

TERUMO MEDICAL PRODUCTS (HANGZHOU) CO., LTD.
M4-9-5 Hangzhou Economic and Technological Development Zone, 310018 Hangzhou, P.R.CHINA
MADE IN CHINA

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All brand names are trademarks or registered trademarks of TERUMO CORPORATION and their respective owners.