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 **TERUMO CORPORATION**  
44-1, 2-CHOME, HATAGAYA, SHIBUYA-KU, TOKYO 151-0072, JAPAN   MADE IN JAPAN

© TERUMO CORPORATION   2019-08

## INDICATION FOR USE

The Heartrail III guiding catheter is intended for cardiac and vascular procedures.  
It is designed to deliver radiopaque media, guide wires, catheters, and therapeutic agents to selected sites in the vascular system.  
The different shapes are designed to selectively engage arteries from access sites such as the radial artery.

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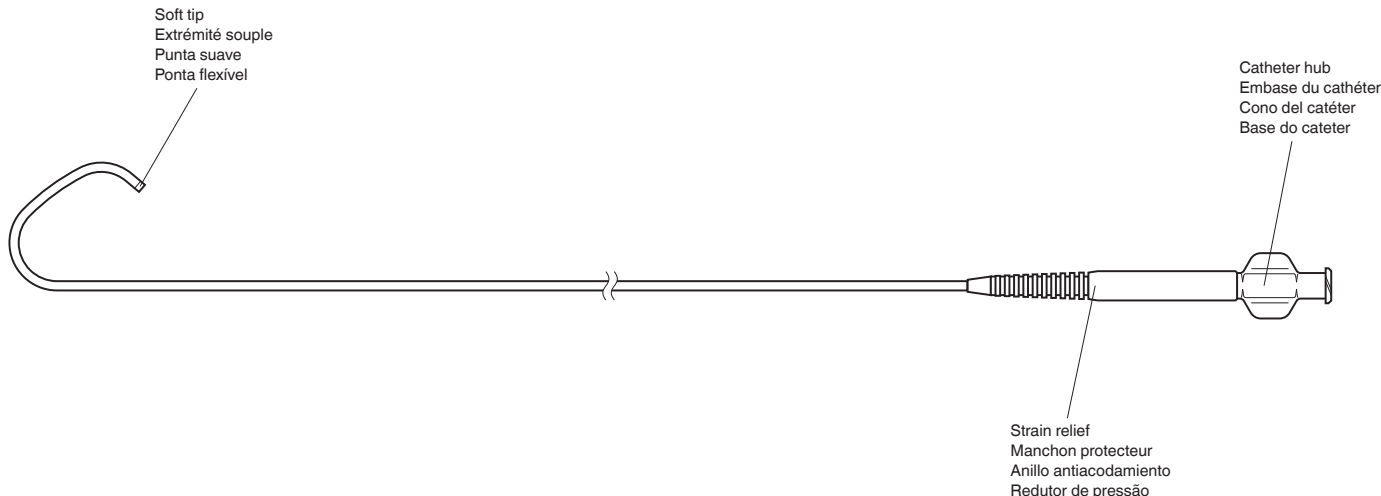
## ENGLISH

### Heartrail III<sup>TM</sup>

Please read all instructions prior to use.

#### CAUTIONS FOR USE

##### 1. CONTRAINDICATIONS

This device is contraindicated for patients who, because of their condition, are contraindicated for PTCA or PTA procedures.  
Do not torque the guiding catheter excessively while kinked. It may cause damage which could result in possible separation along the catheter shaft.  
\* Refer to the contraindications section of the instruction manual accompanying the interventional devices to be used with the catheter.

##### 2.IMPORTANT SAFETY INSTRUCTIONS

###### WARNINGS

- Manipulate the guiding catheter slowly and carefully in the vessel. If any resistance is felt or if torque can not be transmitted properly to the catheter tip, kinking or distortion may occur. In such case, stop manipulating the guiding catheter and determine the cause under fluoroscopy. Manipulating the guiding catheter without determining the cause may result in damage to the vessel, breakage or separation of the catheter. This may necessitate recovery of fragments of the catheter.
- Do not advance the guide wire hastily and/or insert it into the guiding catheter by force when the catheter is bent or twisted. Such manipulations may cause the catheter to rupture/break resulting in damage to the vessels.

###### CAUTIONS

- Confirm that the design and size of the guiding catheter is suitable for the operation and technique to be used.
- Administer appropriate anticoagulant and coronary vasodilator to the patient during the PTCA or PTA procedure.
- Do not use any agents containing organic solvents.
- Do not use oleaginous contrast media which may damage the catheter's hub.

##### 3.COMPLICATIONS

Possible complications of PTCA or PTA include, but are not limited to, the following:

- |                                        |                               |                                          |                                           |
|----------------------------------------|-------------------------------|------------------------------------------|-------------------------------------------|
| •Coronary artery dissection            | • Ischemia due to spasm       | • Bradycardia                            | • Coronary artery injury                  |
| • Ischemia due to long-term dilatation | • Nausea and vomiting         | • Total occlusion of the coronary artery | • Arterial rupture                        |
| • Palpitation                          | • Acute myocardial infarction | • Distal embolization                    | • Behavior disorder                       |
| • Cerebral embolization                | • Artery perforation          | • Hemorrhage                             | • Unstable angina pectoris                |
| • Air embolism                         | • Hypotension                 | • Re-stenosis of the coronary artery     | • Infection and pain at the puncture site |
| • Death                                | • Ventricular fibrillation    | • Hematoma                               | • Intra-vessel thrombosis                 |
| • Arteriovenous fistula                |                               |                                          |                                           |

##### 4.PRECAUTIONS FOR APPLICATION

- This guiding catheter should be used by a physician who is familiar with, and well-trained in, PTCA or PTA techniques.
- The entire operation should be carried out aseptically.

#### DIRECTIONS FOR USE

##### 1.Preparation of the guiding catheter

- Carefully remove the guiding catheter together with the cardboard from the package.
- Flush the guiding catheter lumen by introducing heparinized saline solution through the catheter hub with a syringe.  
**WARNING** Do not attempt to reshape the catheter tip by heating it with a dryer or any other device. The guiding catheter may be damaged.

##### 2.Insertion of guide wire into the guiding catheter

- Insert a guide wire of an appropriate size, previously immersed in heparinized saline solution, into the guiding catheter through the catheter hub and advance it to the distal end of the guiding catheter.

##### 3.Preparation of introduction of the guiding catheter into the vessel

- Use a cut down, Seldinger or percutaneous technique with an introducer sheath of an appropriate size to introduce the guiding catheter into the vessel.  
**WARNING** These techniques must be accompanied by appropriate anticoagulant therapy.

##### 4.Introduction of the guiding catheter into the vessel

- Advance the guiding catheter through the sheath over the guide wire in the vessel.
- Remove the guide wire when the guiding catheter tip reaches the desired position.
- Under fluoroscopy, lead the guiding catheter tip right into the desired site and maintain it in position.

###### WARNINGS

- The guiding catheter may occlude the coronary artery. Care must therefore be taken to avoid complete obstruction of the blood flow.
- Before introducing contrast media or drug, make sure that the guiding catheter is not knotted, kinked or twisted. Injecting contrast media or drug in a knotted, kinked or twisted guiding catheter may cause rupture of the guiding catheter under maximum injection pressure.

###### CAUTIONS

- When inserting the guide wire into the guiding catheter in the vessel, be careful not to cause damage to the guiding catheter when advancing the guide wire through a bend or through the shaped tip of the guiding catheter.
- Maximum injection pressure of The HEARTRAIL III is 700 psi (4826 kPa). When introducing contrast media during the procedure, monitor carefully the injection pressure.

##### 5.Insertion of a dilatation catheter

- Carefully insert a dilatation catheter, prepared for use as recommended by the manufacturer, through the guiding catheter into the vessel.  
**CAUTIONS**
  - Before using a dilatation catheter, make sure its balloon has been deflated adequately.
  - When advancing the tip of the guiding catheter into the coronary artery, do not advance it beyond the distal end of the dilatation catheter. Advancing the guiding catheter beyond the distal end of the dilatation catheter will increase the risk of damaging the coronary artery.
  - Carefully manipulate the catheter if the lesion is around the coronary ostium.

##### 6.Removal of the guiding catheter

- When the intended procedure is completed, remove the guiding catheter tip from the desired position, insert a guide wire of appropriate size into the guiding catheter in such a way that 5 cm of the guide wire tip comes out of the guiding catheter. Remove the guiding catheter from the vessel together with the guide wire.  
**CAUTION** Be careful that advancing the guide wire through a bend or through the shaped tip of the guiding catheter does not perforate or cause damage to the guiding catheter.

#### PRECAUTIONS FOR USE

- This device is sterilized with ethylene oxide gas and is intended for single use only. Do not re-sterilize and/or reuse this device.
- Do not use if the unit packaging or the product has been damaged or soiled. This is an important precaution before you use the product.
- Use immediately after opening the package and dispose of safely following local procedures for the disposal of medical waste.

#### PRECAUTION FOR STORAGE

Avoid exposure to water, direct sunlight, extreme temperature or humidity during storage.

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HEARTRAIL is a trademark of TERUMO CORPORATION.
© : Registered Trademark